



# Infection Control

## MATRIX Full Facemask Respirator

*Personally issued to individual wearers*

When sanitizing is required after contact with a Covid-19 patient, the exposed external surfaces of the MATRIX facemask and P3 filter must be sanitized as instructed below and in accordance with your organisation's infection control policy. The procedure requires the use of Sporidical Wipes that kills all micro-organisms including Covid-19.

### IMMEDIATE SANITIZATION AFTER CONTACT WITH PATIENT




1. Wipe the face seal and inner oronasal mask.



2. Wipe the tabs and buckles of the head harness.



3. Wipe the exposed straps of the head harness.

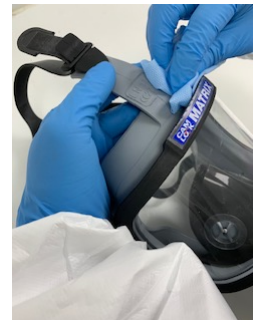

4. Wipe the outer surface of the grey facemask.



5. Wipe the front fitting of the facemask.



6. Wipe behind the P3 filter.



7. With a circular motion, wipe the vents of the P3 filter.



8. Wipe the visor and allow it to air-dry, then buff with a disposable paper towel until the visor is clear.



### PERIODIC: POST USE SANITIZING OF A MATRIX FACEMASK ASSEMBLY

Remove the P3 filter before starting the sanitising procedure. Store the filter for reuse, or discard as necessary.



1. Pull the head harness over the front of the visor and flush the inner mask with clean running water



2. Pour a 50:1 solution of medical, high-level surface disinfectant into the mask and agitate with warm running water for 1 minute.



3. Rinse the inside of the mask with clean running water.



4. Flush the water through the exhalation valve of the mask.



5. Drain excess water by deflecting the reflex edge of the facemask seal.



6. Shake the mask to ensure that no trapped water is retained.



7. Wipe the internal and external surfaces of the mask with a disposable paper towel until the visor is clear.

### ALTERNATIVE PERIODIC SANITIZING PROCEDURE

If a medical, high-level surface disinfectant is not available, an alternative periodic sanitising procedure can be done using a mild liquid detergent in water. Cam Lock recommends Chemgene HLD<sub>4</sub>H medical, high-level surface disinfectant.

### STORAGE

Store the MATRIX mask in a warm, dry place out of direct sunlight and ready for use. Refit the P3 filter prior to use.

**WARNING – CHECK THE MASK PRIOR TO USE**  
**ALWAYS CHECK THE FACE SEAL AFTER DONNING A MATRIX FACE MASK AND BEFORE PROCEEDING TO YOUR WORK LOCATION.**